

# everStick®C&B

## Fibre reinforcement for composite bridges

DISTRIBUTED BY

GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku,  
Tokyo 174-8585, Japan

GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240,  
Interleuvenlaan 33,  
B-3001 Leuven, Belgium  
TEL: +32 16 74 10 00

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
www.gcamerica.com

RESPONSIBLE MANUFACTURER IN CANADA

GC AMERICA INC.  
3737 W. 127th Street, Alsip, IL 60803 U.S.A.

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse,  
#03-05 Singapore 528729  
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.  
1753 Botany Rd, Banksmeadow,  
NSW 2019, Australia  
TEL: +61 2 9301 8200

GC SOUTH AMERICA  
Rua Heliadora, 399,  
Santana - São Paulo, SP, Brasil  
CEP: 02022-051  
TEL: +55-11-2925-0965  
CNPJ: 08.279.999/0001-61  
RESP. T C: MAYARA DE SANTIS RIBEIRO -  
CRO/SP 105.982



**Fibre type:** Silanated E-Glass fibre impregnated with Bis-GMA and PMMA

**Form:** Unidirectional fibre bundle  
**Diameter:** ~1.5-1.7 mm



Keep away from sunlight



Temperature limit

Caution: US Federal law restricts this device to sale by or on the order of a dentist.  
U7608 - 951328

# everStick®C&B

- EN FIBRE REINFORCEMENT FOR COMPOSITE BRIDGES
- DE GLASFASERVERSTÄRKUNG FÜR COMPOSITEBRÜCKEN
- FR FIBRE DE RENFORT POUR BRIDGES COMPOSITE
- IT RINFORZO IN FIBRA PER PONTI IN COMPOSITO
- ES FIBRA DE REFUERZO PARA PUENTES DE COMPOSITE
- NL VEZELVERSTERKING VOOR COMPOSITBRUGGEN
- DA FIBERFORSTÆRKNING TIL KOMPOSITBROER
- SV FIBERFÖRSTÄKNING TILL KOMPOSITBROAR
- PT FIBRA PARA REFORÇO DE PONTES DE RESINA COMPOSTA
- EL ΙΝΕΣ ΥΑΛΟΝΗΜΑΤΩΝ ΓΙΑ ΕΝΙΣΧΥΣΗ ΓΕΦΥΡΩΝ ΑΠΟ ΠΟΛΥΜΕΡΗ ΥΛΙΚΑ
- FI LASIKUITULUJITE KUITULUJITTEISIIN YHDISTELMAMUOVISILTOIHIN



## 1. A surface retained bridge in the anterior region



