

Bonding RPE's Efficiently With GC Fuji ORTHO




Rapid

Palatal

Expander



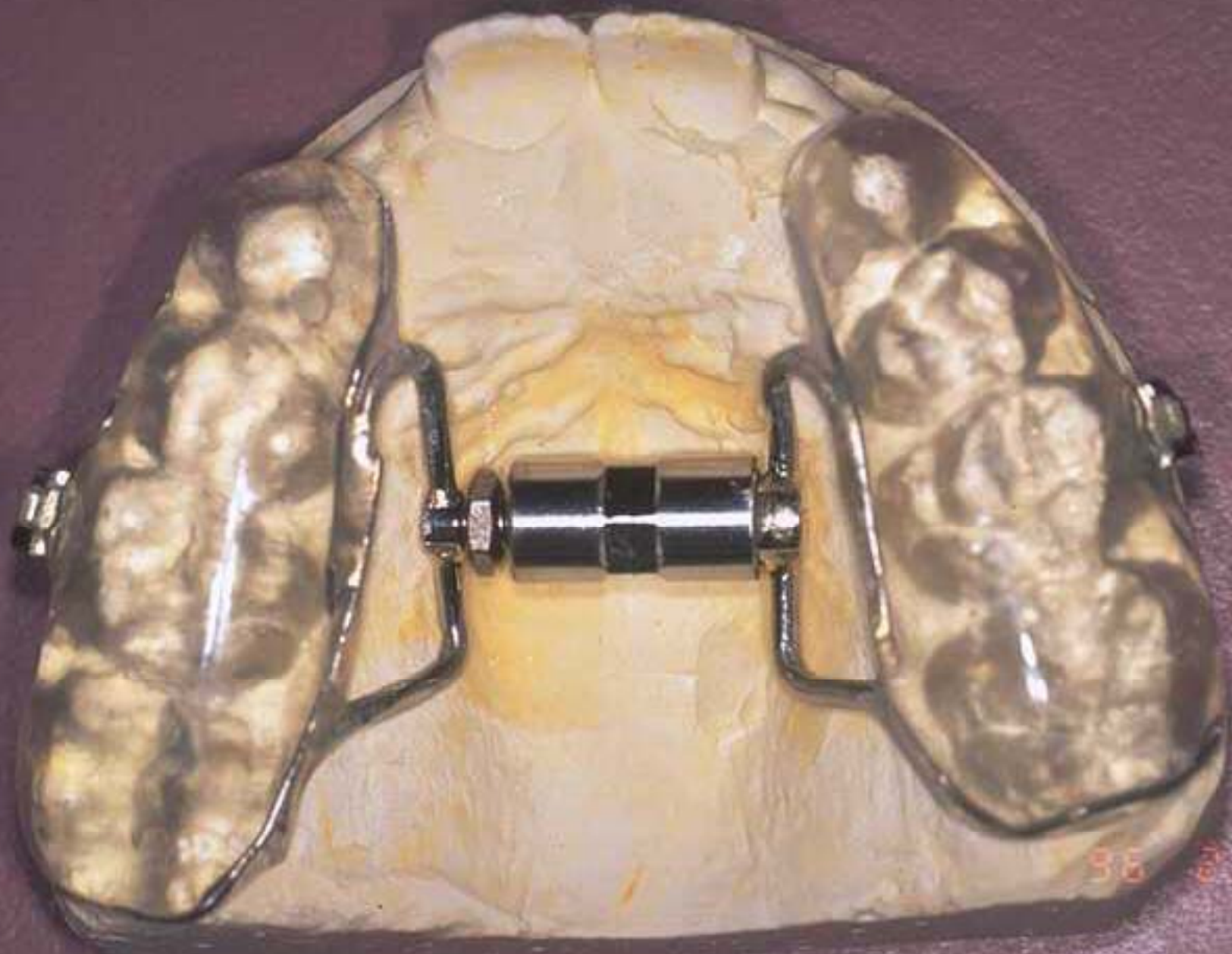




96 2 21

APPLIANCE PREPARATION

GC Fuji Ortho will not bond to acrylic.
Create mechanical retention by
sandblasting or microetching the
internal surfaces of the appliance.





APPLIANCE PREPARATION

For added retention, cut holes throughout the buccal and palatal surfaces.





ENAMEL PRE-TREATMENT

- Using a prophylaxis cup or a brush, clean the bonding surfaces of the teeth with plain (non-fluoridated) pumice and water; rinse with water.
- Apply GC Ortho Conditioner to the bonding surfaces of the teeth for 20 seconds; rinse.

ENAMEL PRE-TREATMENT

To facilitate appliance removal, “block” the occlusal surfaces of the teeth with petroleum jelly.





- 10 DROPS
- 3 PWDN











