Prior to use, carefully read the instructions for use.

EN

GC Tri Plaque ID Gel™

DENTAL PLAQUE DISCLOSING GEL

For use only by a dental professional in the recommended indications.

DESCRIPTION

GC Tri Plaque ID Gel will differentiate among fresh plaque, mature plaque (more than 48 hours old) and strong acid producing plaque by colour to aid in patient oral hygiene education.

CONTRAINDICATIONS

Do not use this material on patients with a sensitivity or allergy to benzoate preservatives.

DIRECTIONS FOR USE

- Apply GC Tri Plaque ID Gel onto the tooth surfaces using a suitable instrument (microbrush, swab or tooth brush). Dispense gel into a dish if you apply on multiple tooth surfaces. Following this, instruct the patient to lightly rinse their mouth with water. Identify the plaque by observing the color of the gel.
 - A pink or red colour on the tooth surface indicates fresh plaque accumulation.
 - A blue or purple colour indicates mature plaque that is at least 48 hours old.
 - A light blue color indicates mature and strong acid producing plaque.

Note:
Clean the tip of the tube by wiping with a sponge or cotton pellet to remove any excess gel. Close the cap immediately.
- Instruct the patient in proper tooth cleaning and thereby remove any remaining plaque and disclosing gel.

STORAGE

Recommended for optimal performance, store in a cool dry low humidity area away from direct sunlight (8-25°C). Shelf life: 3 years from date of manufacture.

PACKAGE

Tube containing 40g (36ml)

CAUTION

- Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.
- Avoid contact with skin, clothing or equipment. In case of contact, immediately wash with water.
- Avoid contact with dental cavity. In case of contact, immediately wash with water. And if the area is stained, the color will gradually disappear.
- This product contains red and blue dyes, which will stain dental restorative materials.
- Do not swallow.
- Do not mix with other products.
- Keep out of reach of children.
- If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives. In this event, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

http://www.gceurope.com or for The Americas http://www.gcamerica.com They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. **DO NOT IMMERSE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 08/2016

'GC.'

DISTRIBUTED BY

GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC EUROPE N.V.

Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.

3737 West 127th Street, Alsip, IL 60803 U.S.A.

TEL : +1-708-597-0900

GC ASIA DENTAL PTE. LTD.

11 Tampines Concourse, #03-05, Singapore 528729

TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.

1753 Botany Rd, Banksmeadow, NSW 2019, Australia TEL: +61 2 9301 9200

MANUFACTURED BY

NIMI CHEMICAL INDUSTRIAL CO., LTD.

4-13 Yokoyama-chou, Kiriyu-shi, Gumma-ken, 376-0055 JAPAN

RESPONSIBLE MANUFACTURER IN CANADA:

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