INSTRUCTIONS FOR USE (IFU)

MI Paste Plus™

Last revised: 07/2017

3200717

MI Paste Plus™

TOPICAL PASTE WITH CALCIUM, PHOSPHATE AND FLUORIDE

For use by or under the supervision of a dental professional in the recommended indications.

Prior to use, carefully read the instructions for use.

EN

MEDICINAL INGREDIENTS

Sodium Fluoride (900 ppm F) 0.2% wt, Casein Phosphopeptide-Amorphous Calcium Phosphate (CPP-ACP) 10% wt

NON-MEDICINAL INGREDIENTS

Water, Glycerol, D-sorbitol, CMC-Na, Propylene glycol, Silicon dioxide, Titanium dioxide, Xylitol, Phosphoric acid 75% aqueous solution, Sodium saccharin, Flavor, Ethyl parahydroxybenzoate, Propyl parahydroxybenzoate, Butyl parahydroxybenzoate

DESCRIPTION

MI Paste Plus is a water based paste containing RECALDENT™ (CPP-ACP) with incorporated fluoride. The level of fluoride is 0.2% wt (900 ppm F), which approximates that in adult strength toothpastes. When CPP-ACP with fluoride is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localizing bio-available calcium, phosphate and fluoride.

RECOMMENDED DOSE

Adults: In dental office using custom tray application or non-tray application using a cotton swab, prophyl cup or a clean finger. At home application, apply twice per day after tooth brushing or as recommended by a dental professional.

Children: 12 years and above: In dental office using custom tray application or non-tray application using a cotton swab, prophyl cup, or a clean finger. At home application apply, twice per day after tooth brushing or as recommended by a dental professional.

IMPORTANT

Saliva will enhance the effectiveness of CPP-ACP and the flavor will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

A topical paste containing bio-available calcium, phosphate and fluoride:

1. Provides extra protection for teeth.
2. Helps neutralize acid challenges from acidogenic bacteria in plaque and from other internal and external acid sources.

CONTRAINDICATIONS

Casein phosphopeptides are derived from milk casein.

Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

DIRECTIONS FOR USE

INDICATIONS

- a. Adults and children 12 years and above, apply twice per day.
- b. As an alternative means of applying fluoride topically in children aged 12 and above.
- c. During orthodontic treatment.
- d. For high risk caries patients.
- e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
- f. For special needs adult patients.

IN-OFFICE APPLICATION

I. Custom tray application:

1. Before use rinse the custom tray thoroughly under running water.
2. Extrude a generous layer of MI Paste Plus into the tray and apply to the upper and / or lower teeth.
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1-2 minutes) avoiding expectoration and delaying swallowing. The longer MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
7. Any residual MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

II. Non tray application:

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply MI Paste Plus to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush.
3. Leave MI Paste Plus undisturbed for a minimum of 3 minutes.
4. Then instruct the patient to use the tongue to spread the remaining MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration and delaying swallowing. The longer MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
5. Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

AT-HOME APPLICATION

III. Application after tooth brushing as recommended by a dental professional:

1. Apply a sufficient amount of MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with MI Paste Plus.
2. Leave MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not swallow. Do not eat, drink or rinse with water for 30 minutes after use.

FLAVORS

Mint (I)

STORAGE

Recommended for optimal performance, store in a cool dry, low humidity area away from direct sunlight (8-25°C)(46.4-77.0°F).

Shelf life: 2 years from date of manufacture.

PACKAGES

Tube containing 40 g (35 mL)

NOTE

MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.

*RECALDENT is a trademark used under license. CPP-ACP Technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

CAUTION

1. Not recommended on patients under the age of 12.
2. Do not swallow.
3. Do not eat, drink or rinse with water for 30 minutes after use.
4. Keep out of reach of children.
5. If more than used for brushing is accidentally swallowed, get medical help or contact a Poison Control Center right away.
6. In case of contact with eyes, flush immediately with water and seek medical attention.
7. In case of spillage on clothes, wash off with water.
8. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician.
9. Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.

Imported by:

Lomas Logistics
4-3520 Laird Road
Mississauga, Ontario L5L 5Z7

NPN 80028772

MANUFACTURED BY
GC-AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Babashi-ku, Tokyo 174-8585, Japan

GC-AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

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