INSTRUCTIONS FOR USE (IFU)

MI Paste™

Last revised: 07/2017

6780717

MI Paste™

TOPICAL PASTE WITH BIO-AVAILABLE CALCIUM AND PHOSPHATE

For use only by a dentist or physician in the recommended indications.

DESCRIPTION

MI Paste™ is a water based topical paste containing RECALDENT™* (CPP-ACP : Casein Phosphopeptide – Amorphous Calcium Phosphate). It is a unique blend of polishing, cleaning and dentinal tubule sealing agents designed for professional application during standard dental practice hygiene procedures. When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue, localizing bio-available calcium and phosphate.

IMPORTANT

Saliva will enhance the effectiveness of CPP-ACP and the flavor will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

RECOMMENDED INDICATIONS

To be used for cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. Secondly, MI Paste™ can be used for the management of tooth sensitivity, post-scaling, root planing and bleaching.

CONTRAINDICATIONS

RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use on patients with a milk protein or hydroxybenzoates allergy.

DIRECTIONS FOR USE

1. Prophy Cup Application

- Following routine prophylaxis for removal of plaque, food debris and stain, ask the patient to rinse.
- Apply a generous layer of MI Paste™ as a final finishing paste to the tooth surfaces using a prophy cup, prophy brush, a gloved finger or in difficult interproximal areas using an interproximal brush.
- Request the patient to hold in the mouth for as long as possible (1-2 minutes) avoiding expectoration and delaying swallowing. The longer MI Paste™ and saliva are maintained in the mouth, the more effective the result.
- Advise the patient not to eat or drink for 30 minutes following application.

Note:

Apply the paste to the teeth - especially at the cervical surfaces where most tooth sensitivity occurs.

II. Custom tray application - Management of tooth sensitivity

- Rinse the custom tray thoroughly under running water.
 - Extrude a generous layer of MI Paste™ into the tray and apply to the upper and / or lower teeth.
 - Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
 - Remove the tray.
 - Instruct the patient to use the tongue to spread the remaining MI Paste™ throughout the mouth. Instruct the patient to retain for as long as possible (1-2 minutes) avoiding expectoration and delaying swallowing. The longer MI Paste™ and saliva are maintained in the mouth, the more effective the result.
 - Ask the patient to rinse any MI Paste™ remaining on the surface. Any residual MI Paste™ can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
 - Any residual MI Paste™ in the tray should be rinsed or brushed off under running water immediately after use.
- Note:
Non tray application –
Apply a generous layer of MI Paste™ to the tooth surfaces as required. Follow Steps 5 and 6 as above.

FLAVORS

Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)

STORAGE

Recommended for optimal performance; store in a cool dry low humidity area away from direct sunlight (8 - 25°C) (46.4 - 77.0°F).

Shelf life: 2 years from date of manufacture.

PACKAGE

40g (35mL) Tube

NOTE

MI Paste™ with its unique RECALDENT™ (CPP-ACP) formulation is available through the dental profession and is not sold in pharmacies, supermarkets or other retail outlets.

*CPP-ACP was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.

RECALDENT™ is a registered trademark used under license. CPP-ACP Technology has related patents pending in Australia, NZ, Europe, Canada and USA.

CAUTION

- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of spillage on clothes, wash off with water.
- If any angioedema symptoms are experienced, discontinue the use of the product and refer to a physician.
- CAUTION: RECALDENT™ is derived from milk casein. Do not use on patients with a milk protein or hydroxybenzoates allergy.
- In case of allergic reaction, stop use, rinse mouth with water and seek medical advice.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available for the Americas at:
<http://www.gcamerica.com/downloads>

Last revised: 07/2017

MANUFACTURED by
GC AMERICA INC.
ALSIP, IL 60803 U.S.A.

DISTRIBUTED by
GC AMERICA INC.
ALSIP, IL 60803 U.S.A.

PRINTED IN THE USA

Prior to use, carefully read
the instructions for use.

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