

HERE'S HOW YOU CAN OBTAIN LONG, SATISFACTORY SERVICE FROM YOUR NEW COE STAINLESS STEEL IMPRESSION TRAY.

COE Stainless Steel Impression Trays are constructed of high quality stainless steel, with handles and rims that are welded, rather than soldered. The welding assures a complete, durable bond. Stainless steel is a long lasting material that will retain its shape even after extended use. Because of this characteristic, rigidity, never use force to modify the shape of the tray. The use of rope wax or compound is an option when adjustments are necessary.

To clean your COE Stainless Steel Tray, remove the bulk of the impression material from the tray. Immerse the tray in a solution of COE TRAY CLEANER and warm water for 15-30 minutes. Stubborn particles of alginate, compound, wax, etc. may then be removed by lightly brushing with a soft bristle brush. Whenever possible, trays should be stored dry.

NOTE

Trays may be routinely sterilized by the following methods:

Autoclave

Chemical-vapor

Immersion (dental sterilants) in Coecide™XL or

Coecide™XL+ glutaraldehyde products.



GC AMERICA INC.

3737 WEST 127TH STREET
ALSIP, ILLINOIS 60803
TEL (800) 323-7063

3590697