# Master all your challenges

*initial*<sup>™</sup>  
Lustre Pastes NF  
Gum Shades

- EN Technical Manual
- DE Technische Arbeitsanleitung
- FR Manuel Technique
- IT Manuale tecnico
- ES Manual técnico
- PT Manual técnico

JUNE 2019

- EN True-to-nature ...
- DE Naturidentisch ...
- FR Proche du naturel ...
- IT Estetica naturale ...

Photoshooting 1 - 4: CDT D. Ibraimi, Switzerland  
Photoshooting 5 - 6: Byrnes Dental Laboratory, UK

EN	DE	FR	IT	Page
<b>Table of Contents</b>	<b>Inhaltsverzeichnis</b>	<b>Table des matières</b>	<b>Indice</b>	
Intended Use	Bestimmungsgemäßer Gebrauch	Utilisation	Campo di impiego	4-5
Description of the Lustre Pastes NF Gum Shades Basic colours Effect colours	Beschreibung der Lustre Pastes NF Gum Shades Basis-Farben Effekt-Farben	Description des Lustre Pastes NF Gum Shades Teintes de base Effets de teinte	Descrizione delle Lustre Pastes NF Gum Shades Colori di base Colori per effetti	6-7
The liquids of the Lustre Pastes NF Gum Shades Set	Die Flüssigkeiten des Lustre Pastes NF Gum Shades Set	Les liquides du coffret Lustre Paste NF Gum Shades	I liquidi del set di Lustre Paste NF Gum Shades	8-9
Lustre Pastes NF Gum Shades Application Basic Step by Step Instruction (Zirconium Frame) Basic Instruction (Non Precious & Ti Dental Frame)	Lustre Pastes NF Gum Shades Anwendung Basis-Farben Step by Step Anleitung (Zirkon-Gerüst) Basis-Farben Anleitung (Nichtedelmetall- / Ti-Gerüst)	Application Lustre Pastes NF Gum Shades Etape par Etape (armature zircon) Etape par Etape (armature dentaire non précieuse / armature Titane)	Applicazione delle Lustre Pastes NF Gum Shades Istruzioni fondamentali passo per passo (frame in zirconia) Istruzioni di base (frame in leg vile / struttura in Titanio)	10-15 16-17
Firing Chart	Brenntabelle	Tableau des températures	Grafico delle cotture	18
Tips and Tricks	Tipps und Tricks	Trucs et astuces	Consigli e accorgimenti	19
Facilities of the Lustre Pastes NF Gum Shades	Möglichkeiten mit Lustre Pastes NF Gum Shades	Exemples d'application des les Lustre Pastes NF Gum Shades	Esempi di applicazioni delle Lustre Pastes NF Gum Shades	20-21
FAQ	FAQ	FAQ	Domande più frequenti	22-23

# GC Initial IQ Lustre Pastes NF Gum Shades

## EN Intended Use

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### EN Paint-on your gum shades

GC Initial IQ Lustre Pastes NF Gum Shades are 3-dimensional ceramic stains developed to enable dental technicians to create a lifelike transition from the crown margin to the gingiva just by painting the gum shades. With a continuously growing demand for aesthetic implant-supported superstructures, there has also been a growing desire among dental technicians for a ceramic staining approach for the soft tissue zones. With the new unique Lustre Pastes NF Gum Shades Set, you can from now on bring not only the white, but also the red aesthetics of your crowns and bridges to life. These unique gum coloured Lustre Pastes allow an individual adaptation to match natural gingival shades offering a natural harmony. With their optimal match to both "low" and "high" CTE ceramics, the system is compatible with the complete GC Initial ceramic range. Available in a ready to use consistency, the Lustre Pastes NF Gum Shades are based on fine ceramic particles and can be applied in a thicker layer where they will exhibit unsurpassed nature analogue gum reproduction and a very natural glaze to your restorations.

### Features

- One kit for the complete GC Initial ceramic range.
- No complex layering or extra firing cycles required. Fast – Easy – Secure.
- Available in ready to use jars for a more convenient application.
- Diluting and Refresh liquids to keep the optimal consistency until the end.
- Five gum coloured pastes with an ideal thixotropy for easy and precise application.
- Four mixable Gum Modifiers allowing unlimited creativity to copy natural gingival harmony.

### Indications

The Lustre Pastes NF Gum Shades are a one for all solution for the complete Initial line from GC. It's possible to use the Lustre Pastes NF Gum Shades in a wide CTE – Range in order to work with nearly all types of ceramic. These Lustre Pastes NF Gum Shades represent a modern alternative for conventional staining of gingival areas. Due to the unique properties it's just a question of minutes to individualize the gingival areas of your crown and bridge work.

**Keep in mind: What you paint (on) is what you get (after firing)**

**GC Initial IQ Lustre Pastes NF Gum Shades Set** Contents: 1x Base Light, 1x Base Dark, 1x Intensive Violet, 1x Intensive Red, 1x Intensive Cream, 4x Modifier (white, red, blue, brown), 1x Diluting Liquid, 1x Refresh Liquid, 1 mixing dish, 2 brushes, 1x Technical Manual, 1x Colour Chart

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LP-M2  
Modifier Red

**EN Indications:**

The modifiers can be used to intensify the single Lustre Pastes Gum Shades allowing an unlimited reproduction of gum colours.

**Instructions for use:**

The different modifiers can be added & mixed directly with the Lustre Pastes Gum Shades. The desired colour can visually be controlled.

When adding modifiers to the Lustre Pastes Gum Shades there is no need to adapt the firing parameters.

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#### EN Lustre Paste 'Diluting Liquid':

The 'Diluting Liquid' is a special liquid to change the flowability of the Lustre Pastes. However, it is recommended to obtain a gel/paste like consistency of the Lustre Pastes for best results.  
Attention:

- If the Lustre Pastes are too much diluted, the result is a reduction in the gloss of the paste after the firing; hence, the desired result may not be achieved in one glaze firing.
- When needed, dilute the desired amounts of the pastes in a separate plate with Diluting Liquid, never dilute directly in the jars.

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**EN Lustre Paste 'Refreshing Liquid':**

The 'Refreshing Liquid' is used to preserve the thixotropic property of the pastes. This liquid is used to recover dried out pastes to an optimum, gel like consistency without changing the physical properties. The Refresh Liquid can be used both in the jars and mixed on a separate plate.

Important: Please shake the Refresh Liquid well before use!

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EN After the second firing the restoration can be shaped & contoured in the usual way. Before applying the Lustre Coating, the ceramic restoration is shaped and contoured using adjusted diamond burs. After polishing with a ceramic polisher the finished Crowns & Bridges are sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure.

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EN To intensify the areas between the tooth roots, e.g. Lustre Paste G-36 can be used.

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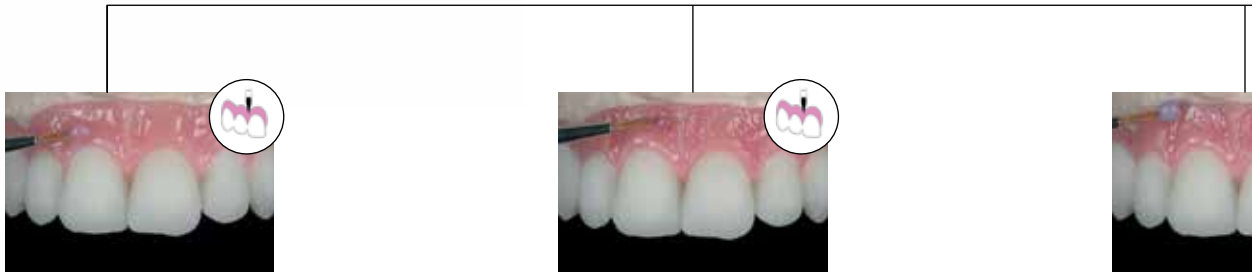
EN For more detailed instructions, please refer to the dedicated Technical Manuals / IFU of each of the below GC Initial Lines:

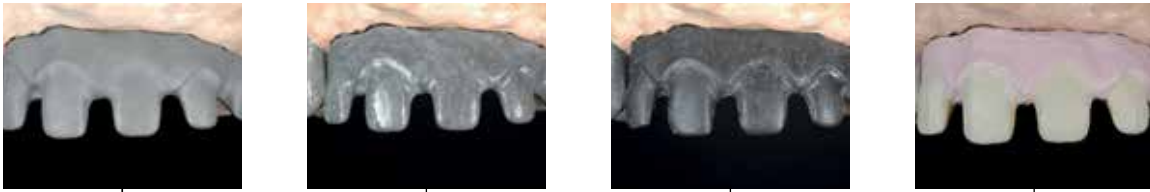
- GC Initial MC/LF
- GC Initial Ti
- GC Initial Ti Bonder
- GC Initial IQ Layering-over-Metal
- GC Initial IQ Press-over-Metal
- GC Initial INmetalbond

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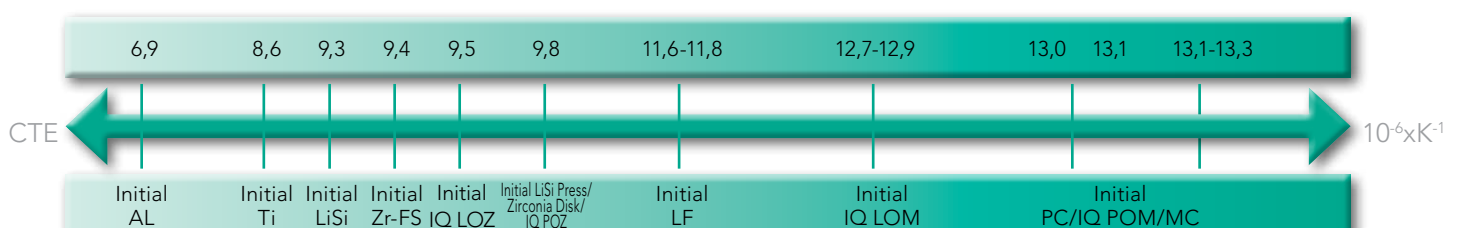
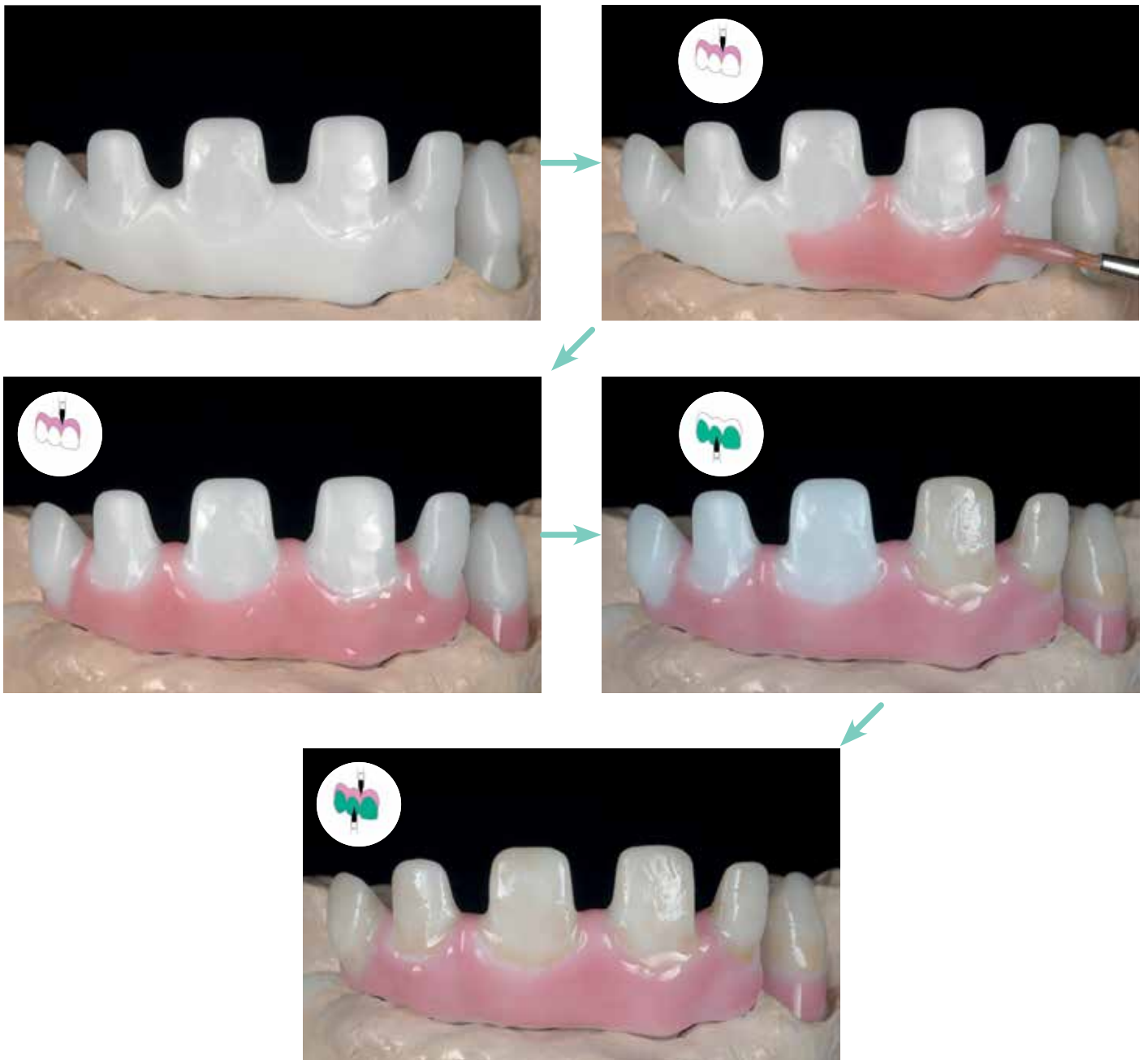
DE FAQ

FR FAQ

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## EN Basic Rule: What you paint (on) is what you get (after firing)

### Q1. How can I dilute (adapt the consistency of) the Lustre Pastes NF Gum Shades?

Use the Lustre Paste NF Diluting Liquid to adapt the consistency.

Do NOT use water to dilute. LP may not come in contact with water (see page 8).

### Q2. What can I do when the consistency of the Lustre Pastes NF Gum Shades is too dry?

Use the Refresh Liquid to recover the Lustre Pastes in case of dryness (see page 9).

### Q3. How do we change the texture of the Lustre Pastes NF Gum Shades?

The requested surface texture/smoothness of the Lustre Pastes NF Gum Shades layer can be changed by soft vibration or condensing the restoration.

### Q4. I do not have a shiny appearance after firing the Lustre Pastes NF Gum Shades, what is the cause?

Check the firing temperatures, increase the final temperature. When the Lustre Pastes NF Gum Shades are heavily diluted or applied in a too thin layer the glossy effect is tempered, so apply a more thick layer/dilute less the pastes.

### Q5. Can I apply more than one layer of Lustre Pastes NF Gum Shades?

Yes, you can do a second firing, keeping the same firing temperatures.

### Q6. Can I mix the Lustre Pastes NF Gum Shades with other Initial powders?

You can use the GC Initial Modifier powders to intensify the Lustre Pastes NF Gum Shades.  
Mix the required amount of Modifier powder with the LP Dilution Liquid and mix it with the required LP or directly mix the Modifier powders with e.g. the Lustre Pastes NF Gum Shades.

### Q7. Are the Lustre Pastes NF sufficient abrasion resistant?

Yes, the LP NF Gum Shades are a special mixture of different types of low fusing ceramic particles, stains & glaze mixture offering lifelike aesthetics.

### Q8. The glaze firing is blarily, spotted or cracked?

The Lustre Pastes NF Gum Shades came partly or completely in contact with water.

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GC Initial IQ Lustre Pastes NF Set - V-Shades

GC Initial IQ Lustre Pastes NF Set - Gum Shades

#### **EN Medical Device Regulation Requirement**

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)  
In this way you will contribute to improve the safety of this product.

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Handwriting practice lines consisting of 25 horizontal dotted lines.

Handwriting practice lines consisting of multiple horizontal dotted lines across the page.

 EU: GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240  
Interleuvenlaan 33, B-3001 Leuven  
Tel. +32.16.74.10.00, Fax. +32.16.40.02.14  
info.gce@gc.dental, www.gceurope.com

RESPONSIBLE MANUFACTURER IN CANADA  
GC AMERICA INC.  
3737 W. 127th Street, Alsip, IL 60803 U.S.A.

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
Tel: +1-708-597-0900  
www.gcamerica.com

GC Germany GmbH  
Seifgrundstr. 2, D-61348 Bad Homburg  
Tel. +49.61.72.99.59.60, Fax. +49.61.72.99.59.66.6  
info.germany@gc.dental, www.germany.gceurope.com

GC ITALIA S.r.l.  
Via Calabria 1, I-20098 San Giuliano Milanese  
Tel. +39.02.98.28.20.68, Fax. +39.02.98.28.21.00  
info.italy@gc.dental, www.italy.gceurope.com

GC UNITED KINGDOM Ltd.  
Coopers Court, Coopers Court, Newport Pagnell, UK-Bucks. MK16 8JS  
Tel. +44.1908.218.999, Fax. +44.1908.218.900  
info.uk@gc.dental, www.uk.gceurope.com

GC FRANCE s.a.s.  
8 rue Benjamin Franklin, F-94370 Sucy en Brie Cedex  
Tel. +33.1.49.80.37.91, Fax. +33.1.45.76.32.68  
info.france@gc.dental, www.france.gceurope.com

GC IBÉRICA - Dental Products, S.L.  
Edificio Codesa 2  
Playa de las Américas, 2, 1º, Of. 4, ES-28290 Las Rozas, Madrid  
Tel. +34.916.364.340, Fax. +34.916.364.341  
comercial.spain@gc.dental, www.spain.gceurope.com

GC AUSTRIA GmbH  
Tallak 124, A-8103 Rein bei Graz  
Tel. +43.312.45.40.20, Fax. +43.312.45.40.20.40  
info.austria@gc.dental, www.austria.gceurope.com

GC Europe NV - Benelux Sales Department  
Researchpark Haasrode-Leuven 1240  
Interleuvenlaan 33, B-3001 Leuven  
Tel. +32.16 74.18.60  
info.benelux@gc.dental, http://benelux.gceurope.com

GC EUROPE N.V. - East European Office  
Siget 19b, HR-10020 Zagreb  
Tel. +385.1.46.78.474, Fax. +385.1.46.78.473  
info.eeo@gc.dental, www.eeo.gceurope.com

GC NORDIC AB - Finnish Branch  
Bertel Jungin aukio 5 (6. kerros), FIN-02600 Espoo  
Tel. +358 40 9000 757  
info.finland@gc.dental, www.finland.gceurope.com

GC NORDIC AB - Danish Branch  
Scandinavian Trade Building  
Gydevang 39-41, DK-3450 Allerød  
Tel: +46 768 544 350  
info.denmark@gc.dental, http://nordic.gceurope.com

GC NORDIC AB  
c/o Lundin Revisionbyrå  
Erik Dahlbergsgatan 11B - S-411 26 Göteborg – Sweden  
Tel. +46 8 410 344 90, Fax. +46 8 555 788 05  
info.nordic@gc.dental, www.nordic.gceurope.com

GC AUSTRIA GmbH - Swiss Office  
Bergstrasse 31, CH-8890 Flums  
Tel. +41.81.734.02.70, Fax. +41.81.734.02.71  
info.switzerland@gc.dental, www.switzerland.gceurope.com

GC AUSTRALASIA DENTAL PTY LTD  
1753 Botany Rd, Banksmeadow NSW 2019, Australia  
Tel: +61-2-9301 8200, Fax: +61-2-9316 4196

GC SOUTH AMERICA  
Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL  
CEP: 02022-051 - Tel: +55-11-2925-0965 - CNPJ: 08.279.999/0001-61  
RESP. TÊC: Mayara de Santis Ribeiro - CRO/SP 105.982

GC ASIA DENTAL PTE. LTD.  
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541  
Tel: +65 6546 7588