Prior to use, carefully read the instructions for use

GB

NATURE-CRYL™ MC

Microwave Cured Denture Base Resin

For use only by a dental professional in the recommended indications.

DESCRIPTION:

NATURE-CRYL MC is a denture base material that has been specially designed to be cured using the microwave technique allowing rapid cure cycles and resulting in dense, strong dentures that exhibit excellent fits.

Type 5- Microwave cured materials per ISO1567

RECOMMENDED INDICATIONS:

Fabrication of full and partial dentures.

WARNING:

Contains methyl methacrylate. In susceptible individuals, sensitization to the product cannot be excluded. Use of the product should be discontinued if allergic reactions are observed.

CAUTION:

- Flammable Liquid. Do not use near open flames or sources of ignition. Close the container immediately after use.
- Avoid contact with skin and eyes. In case of contact with eyes, flush immediately with water and seek medical attention.
- Avoid prolonged contact with the liquid and unpolymerized dough and inhalation of monomer (liquid) fumes/vapors.
- Refer to the MSDS for additional health and safety information.
- Do not mix with other resins.

STORAGE:

NATURE-CRYL MC Powder and Liquid should be stored in a cool (60-80°F, 15-27°C), dark, and dry place away from heat and sources of ignition or from sunlight.

SHELF LIFE:

NATURE-CRYL MC Powder: 3 years from the date of manufacture
NATURE-CRYL MC Liquid: 3 years from the date of manufacture

INSTRUCTIONS FOR USE:

Preparatory Work (Microwave Flask)

- Coat the inside of the flask with a thin film of Vaseline or similar lubricant.
- Place the model with the denture wax up into the lower part of the flask and fill with a 50/50 mix of plaster and stone.
- After the gypsum has hardened apply Vaseline to the gypsum surface
- Fill up the top part of the flask with a plaster/stone mix. Close the flask with the top deck. NOTE: to avoid plaster running into the bolt holes use the bolts to close the flask.
- After the Gypsum has hardened, the wax is softened. Be careful not to melt the wax. The softened wax should be removed in one piece. NOTE: When utilizing the microwave flask, it is permissible to place the flask in the microwave oven for one minute on high to generate enough heat to soften the wax prior to removal.
- After wax removal allow the gypsum surface to cool for about 5 minutes. This allows the steam to rise for better bonding of the separator. Next apply the separator (ex: MULTI-SEP™ or COE-SEP™) to the gypsum surface and allow to dry.

Mixing:

Equipment required: mixing jar with cover, powder and liquid mixing vials or a balance and/or graduated cylinder.

| Proportions | (w/w) | (w/v) | (v/v) |
|----------------|-------|--------|-------|
| Powder: | 20 g | 20 g | 30 cc |
| Liquid: | 8 g | 8.6 mL | 9 mL |

Be sure all measuring and mixing utensils are clean, and dry.

- Carefully measure the desired amount of liquid and pour it into the mixing jar. Note: NATURE-CRYL MC comes with a powder scoop and a 4 mL liquid dropper graduated in 1 mL increments. The average denture needs 2 scoops of powder and 8 mL of liquid. (20 g powder and 8.6 mL liquid, 100g / 43 mL powder liquid ratio) *Warning: The liquid is highly flammable. Keep it away from open flames or sources of ignition.*
- Shake powder bottle to redistribute fibers and then carefully measure the desired amount of powder.
- Carefully pour the corresponding quantity of powder over the liquid while tapping the mixing jar until the excess liquid disappears. Do not stir this mixture with a spatula to avoid entrapping air bubbles.
- Close the lid of the mixing jar and leave for about 15-20 minutes at 74° F (23° C). At this point the NATURE-CRYL MC reaches dough stage.

Packing:

- The flask should be at room temperature. Remove the dough from the mixing jar in one piece to avoid entrapping air bubbles. It is recommended not to touch the dough directly because some individuals may be allergic to the liquid. Use a polyethylene sheet. Close the microwave flask. Trial closures are repeated 2 to 3 times until no flash is observed thereby avoiding open vertical dimension (= sur-occlusion)
- After the final trial packing fit the flask back together and insert the bolts and nuts. Apply pressure using a flask press and uniformly tighten the bolts being careful not to over tighten.
- Cure NATURE-CRYL MC in the microwave oven for 3 minutes. For partial dentures with metal frameworks it is recommended that you cure for 1.5 minutes and then turn the flask over and cure for the final 1.5 minutes. This is required because the metal may block the microwave radiation. Use the following guidelines for the power setting of the microwave oven that you are using.

| Microwave rated Wattage | Recommended Power Setting |
|-------------------------|---------------------------|
| 500 | 100% |
| 1000 | 50% |

Failure to adjust the power setting may cause porosity in the denture.

Cooling:

- Bench cool the flask for 30 minutes after irradiation then place in cold water to complete the cooling cycle (approximately 20 minutes).
- Follow conventional methods for deflasking the denture. This procedure generally is accomplished using plaster-snips or a plaster knife.

Finishing and Polishing:

- Finish and polish using standard technique.
- Do not generate excessive frictional heat when finishing and polishing. High temperatures can cause distortion of the denture base.
- Important- Keep the finished denture in water until it is inserted in the patients mouth.
- Instruct the patient to keep denture in water during any period when it is not being worn to prevent it from drying out. This will ensure better conformation to the mouth structures.

Repairs:

It is recommended that repairs be made using auto polymerizing (cold cure) resins such as NATURE-CRYL POUR in order to keep dimensional changes in the repaired denture to a minimum. Be sure that plaster surfaces underlying the repair area are protected by a tinfoil substitute. For maximum strength and density, repairs made with cold cure resins should be cured under pressure in an ACRI-DENSE™ Curing Unit according to the instructions furnished with the unit. If it is necessary to heat cure repairs, the curing temperature should never exceed 165°F(74°C). DO NOT BOIL.

SHADES:

Powder available in 7 Shades

(COE Light Fibered +, COE-LOR™ Mild, COE-LOR™ Moderate, COE-LOR™ Heavy, Original, Light Reddish Pink, Light, Clear)

PACKAGES:

NATURE-CRYL MC 1lb. (454g) Powder only
NATURE-CRYL MC 1:1 Kit 1lb. (454g) Powder, 8 oz. (237mL) Liquid
NATURE-CRYL MC 5lbs. (2.27kg) Powder only
NATURE-CRYL MC 8oz. (237mL) Liquid only
NATURE-CRYL MC 1qt. (946mL) Liquid only

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MADE IN THE U.S.A.
PRINTED IN U.S.A.