## G-ænial Sculpt™

LIGHT-CURED UNIVERSAL NANO HYBRID COMPACTABLE COMPOSITE

For use only by a dental professional in the recommended indications.

### INDICATIONS

- Direct restorative for class I, II, III, IV, V and V cavities.
- Direct restorative for wedge-shaped defects and root surface cavities.
- Direct restorative for veneers and diastema closure

### CONTRAINDICATIONS

- Direct pulp capping
- Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

### DIRECTIONS FOR USE

- Shade Selection
 

Clean the tooth with pumice and water. Shade selection should be made prior to isolation.

Select the appropriate shade from 17 shades of A1, A2, A3, A3.5, A4, B1, B2, B3, C3, AO1, AO2, AO3, BW, XBW, CV, JE, AE.

A, B, C shades are based on the Vita® classical shade guide.

\*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany
2. Cavity Preparation
 

Prepare cavity using standard techniques. Dry by gently blowing with oil free air. Rubber dam is recommended to isolate prepared tooth from contamination with saliva, blood or sulcus fluid.

Note:

For pulp capping, use calcium hydroxide.
3. Bonding Treatment
 

For bonding G-ænial Sculpt to enamel and/or dentin, use a light-cured bonding system such as G-Premio BOND, G-ænial Bond, G-BOND, GC Fuji BOND LC. Follow manufacturer's instructions.
4. Placement of G-ænial Sculpt
  - 1) Dispensing from a Unitip
 

Insert the G-ænial Sculpt Unitip into the Unitip APPLIER II or equivalent. Refer to the applier manufacturer's instructions for use.

Remove the cap and extrude material directly into the prepared cavity. Use steady pressure.
  - 2) Dispensing from a syringe
 

Remove syringe cap and dispense material onto a mixing pad.

Place the material into the cavity using a suitable placement instrument. After dispensing, screw plunger counterclockwise by a half to full turn to release residual pressure inside the syringe. Replace cap immediately after use.

Note:

    1. Material can be applied in a single shade layer to achieve aesthetic restorations using Universal shades. For details, refer to the Clinical Hints.
    2. Material may be hard to extrude immediately after removing from cold storage. Prior to use, let stand for a few minutes at normal room temperature.
    3. After dispensing, minimize exposure to ambient light. Ambient light can shorten the manipulation time.


### Clinical Hints

- In the case of small cavities
 

Restore using a one shade technique. In most cases the use of one Universal shade alone will be sufficient. In cases where a higher degree of translucency is needed, one of the Enamel shades can be selected.
- In the case of large and/or deep cavities

In most cases a multi shade layering technique will give the best aesthetic results. To block out the cavity or to mask discolored dentin, select an appropriate Opaque shade and continue to build up with a Universal shade. For optimal esthetics use an Enamel shade as the final composite layer. In the case of deep posterior cavities, a flowable composite such as GRADIA DIRECT Flo / LoFlo or G-ænial Flo or a glass ionomer cement such as GC Fuji LINING LC (Paste Pak) or GC Fuji IX GP can be used on the cavity floor instead of an opaque shade.

Examples of Clinical Applications (Clinical Hint)

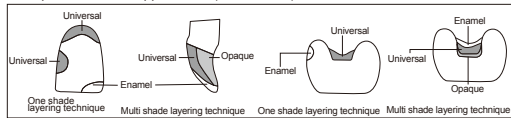### 5. CONTOURING BEFORE LIGHT CURING

Contour using standard technique.

### 6. LIGHT CURING

Light cure G-ænial Sculpt using a light curing unit. Keep light guide as close as possible to the surface. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

Irradiation Time and Effective Depth of Cure	9 sec. (Plasma arc) (2000mW/cm <sup>2</sup> )
	10 sec. (High Power LED Light) (>1200mW/cm <sup>2</sup> ) 20 sec. (Halogen/LED) (700 mW/cm <sup>2</sup> )
A01, A02, A03, A3.5, A4, CV	1.5 mm
A1, A2, A3, B1, B2, B3, C3, BW, XBW	2.0 mm
JE, AE	2.5 mm

#### Note:

1. Material should be placed and light cured in layers. For maximum layer thickness, please consult above table.
2. Lower light intensity may cause insufficient curing or discoloration of the material.

### 7. Finishing and Polishing

Finish and polish using standard techniques.

### SHADES

A1, A2, A3, A3.5, A4, B1, B2, B3, C3, AO1, AO2, AO3, XBW, BW, CV, JE, AE

### STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39.2-77.0°F) (Shelf life : 3 years from date of manufacture)

### PACKAGES

1. Unitips
  - a. Pack of 20 tips (each in 9 shades) (0.16mL per tip)
 

A1, A2, A3, A3.5, A4, B1, B2, B3, C3
  - b. Pack of 10 tips (each in 8 shades) (0.16mL per tip)
 

A01, AO2, AO3, XBW, BW, CV, JE, AE

NOTE:

Weight per unitip : 0.3g
2. Syringes
  - 1 Syringe (in 17 shades) (2.0mL per syringe)
 

NOTE:

Weight per syringe : 3.8g

### CAUTION

1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. Take care to avoid ingestion of the material.
4. Prior to starting the filling process, partially extrude paste outside of the patient's mouth and away from the patient to ensure paste is flowing properly.
5. Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
6. For infection control reasons, Unitips are for single use only.
7. The barrel of the dental composite applier needs to be disinfected between patient uses. Please use an alcohol wipe.
8. Wear protective eye glasses during light curing.
9. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
10. Do not mix with other similar products.
11. Avoid getting material on clothing.
12. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge, or cotton pellet before light curing.
13. Do not use G-ænial Sculpt in combination with eugenol containing materials as eugenol may hinder G-ænial Sculpt from setting.
14. Dispose of all wastes according to local regulations.
15. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
16. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

http://www.gcamerica.com  
They can also be obtained from your supplier.

**CLEANING AND DISINFECTING:**  
**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.  
**DO NOT IMMERSÉ.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised: 11/2017

Rx Only

**GC**

MANUFACTURED by  
**GC DENTAL PRODUCTS CORP.**  
 2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan  
 RESPONSIBLE MANUFACTURER IN CANADA:  
**GC AMERICA INC.**  
 3737 West 127th Street, Alisp, IL 60803 U.S.A.

DISTRIBUTED by  
**GC CORPORATION**  
 76-1 Haseguma-cho, Itabashi-ku, Tokyo 174-8585, Japan  
**GC AMERICA INC.**  
 3737 West 127th Street, Alisp, IL 60803 U.S.A.  
 TEL: +1-708-597-0900  
 www.gcamerica.com

**GC SOUTH AMERICA**  
 Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL  
 CEP: 02029-051 - TEL: +55-11-2925-0965  
 CNPJ: 08.279.999/0001-61  
 RESP. TEC: Mayara de Santis Ribeiro - CRO/SP 105.982

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