



34 CLINICAL EVALUATORS

421 TOTAL USES

96% CLINICAL RATING

Key features: Resin-reinforced glass ionomer cement • Tack-cure feature
• Syringe delivery system

Description

FujiCEM® Evolve is an innovative resin-reinforced glass ionomer cement with:

- Trustful performance to lute any retentive zirconia, lithium disilicate, or metal-base restorations
- Enjoyable convenience thanks to a modern delivery
- High radiopacity
- Locking tip

Indications

- High-strength, all-ceramic (zirconia) crowns and bridges
- Composite inlays, onlays, crowns, and bridges
- All-ceramic inlays
- Metal, ceramic and fiber posts
- Metal and porcelain-fused-to-metal (PFM) crowns and bridges

Unique Attributes

- Highest bond strength to zirconia among RMGI cements tested at DENTAL ADVISOR biomaterials lab
- High radiopacity
- Three-second tack cure
- 4 minute 30 second set time after seating
- No primer necessary when cementing retentive zirconia, lithium disilicate, or PFM restorations
- Moisture tolerant



Clinical Tips


- Make sure the tip is locked in properly before use.
- Remove cement from contact points first before it sets too much.
- Bench test tack feature before using the first time intraorally to determine the time needed with your curing light.
- New tip design allows for automix or hand mixing material.

**"CONVENIENT
AUTO-MIX
SYSTEM."**


Evaluators' Comments

- "Easy to use and easy to clean excess cement with tack-cure feature."
- "I liked the way it dispensed much better than what I use now."
- "Works just as well as the FujiCem 2 that I have used for years with unparalleled success."
- "The color is easy for visualization."
- "Sensitivity is low for patients."
- "The newly designed mixing tip was cumbersome to place and I had a tip pop off when trying to extrude."
- "It took two cures to get the material to tack with our light."

Evaluation Summary:

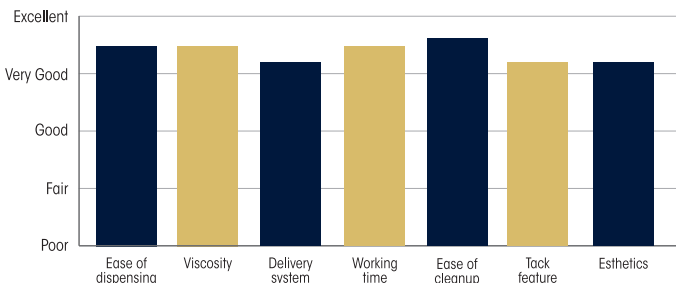

Compared to Competitive Products:

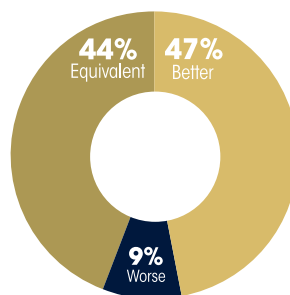

Percentage of Consultants Who Would:

- 94%** Recommend to a colleague
- 6%** Not recommend
- Consultants Who Would Want to Stock in Office:**
- 33%** Yes, instead of current material
- 49%** Yes, in addition to current material
- 9%** No; however, might want for certain cases