

# COE® ADHESIVE

## Directions:

1. Using the brush supplied in package, paint a thin layer of COE Adhesive directly on the tissue surface of the tray.
2. Allow the COE Adhesive to dry thoroughly. A mild stream of air will hasten drying.

## Important...

If COE Adhesive thickens, use methyl ethyl ketone as a thinner. Some separation is normal and will not affect adhesive properties. Shake to thoroughly mix contents. COE Adhesive is flammable and should not be used near an open flame. Inhalation of the fumes should be avoided. Keep bottle tightly closed.



GC AMERICA INC.

3737 WEST 127TH STREET  
ALBANY, ILLINOIS 60803  
TEL (708) 597-0600  
FAX (708) 371-5103