<span>Prior to use, carefully read the instructions for use.</span>	<span><span><span><span></span></span></span><b>EN</b></span>
---	---

# LINING CEMENT

RADIOPAQUE GLASS IONOMER LINING CEMENT

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**  
As a base or liner in prepared cavities.

**CONTRAINDICATIONS**

- Pulp capping.
- Avoid use of this product in patients with known glass ionomer cement allergies.

**DIRECTIONS FOR USE**

1. POWDER AND LIQUID DISPENSING

- a) The standard powder to liquid ratio is 1.2 g/1,0 g.
  - 1 level scoop of powder to 1 drop of liquid.
Note:
For a base, a heavier consistency is desirable. Compress powder in the scoop before levelling.
- b) For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- c) Hold the liquid bottle vertically and squeeze gently.
- d) Close bottles immediately after use.

2. MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into two equal parts. Mix the first portion with all of liquid for 5-10 seconds.

Incorporate the remaining portion and mix the whole thoroughly for 10-15 seconds.

3. RESTORATIVE TECHNIQUE

- a) Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
- b) Wash and dry but **DO NOT DESICCATE**. CAVITY CONDITIONER or DENTIN CONDITIONER is recommended to remove the smear layer.
- c) Mix the required amount of cement.
- d) Remove surface moisture, but **DO NOT DESICCATE**.
- e) Transfer cement to the preparation using a syringe or suitable placement instrument, covering dentine up to the dentino-enamel junction.
- f) Allow cement to set (about 4 minutes).
- g) Proceed with standard composite or amalgam restorative techniques.

**STORAGE**

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

**PACKAGES**

- Bottle of 15 g powder with measure
- Bottle of 15 g (12 mL) liquid
- 1-1 pkg. : 15 g powder, 15 g (12 mL) liquid, powder spoon, plastic spatula, mixing pads

**CAUTION**

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT** mix powder or liquid with any other glass ionomer components.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gcamerica.com
They can also be obtained from your supplier.

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